

# PROFESSIONAL PROCEDURE GUIDE



**A1CNow+**

**A1CNow+**



# PROFESSIONAL PROCEDURE GUIDE

## Preparation

### 1 BEFORE YOU BEGIN

- Run the test with all parts of the test kit at the same temperature within the specified range.
- If the kit has recently been at high temperatures (above 82°F) or in the refrigerator, keep the kit at room temperature for at least one hour before use.
- Avoid running the test in direct sunlight, on hot or cold surfaces, or near sources of heat or cold.
- Quality control materials should be used to confirm the test kit is working properly. Refer to the product insert for information on when to run controls.

#### Monitor (back)

**Complete test within 15 minutes.**

**Base (do not remove!)**

**Pouched Test Cartridge**

**Blood Collector**

**Sampler Body**

**Sample Dilution Kit**

**Wait to Open**

**LOT 0628216**

**EXP 21 SEP 07**

**LOT 0628216**

**2007-09-21**

**CODE A1**

**Lot Number and Dating Label**

**Ensure Lot #'s match**

**0628216**

## Blood Collection

### 2 COLLECT BLOOD

**Fingerstick**

Use your own lancet device to draw blood

Gently touch blood drop to fill

**Or**

**Venous Draw**

Mix blood well before testing

Collect blood from a slide

45°

**Too little add more blood**

**Just right**

**Too much wipe away excess**

## Blood Dilution

### 3 INSERT BLOOD COLLECTOR

Fully insert Blood Collector into Sampler Body

Push firmly!

Keep pushing!

Twisting motion helps

**Not fully inserted Keep pushing!**

**Fully inserted**

### 4 SHAKE

Shake well 6-8 times This will mix the blood with the solution

Stand Sampler on table while preparing Cartridge

### 5 INSERT CARTRIDGE

**2 Test Cartridge**

**OPEN NOW**

Use within 2 minutes.

“Click” Test Cartridge into place

Do Codes Match? \*

Monitor and Test Cartridge codes must match

\* If not, Call Technical Support at 1-877-212-4968 x1

## Blood Testing

### 6 PREPARE SAMPLER

WAIT for SMPL to display

Ready for Sampler

Remove Base

Ensure Monitor is on level surface

### 7 DISPENSE SAMPLE INTO CARTRIDGE

Push down completely to dispense diluted sample Remove quickly

Do not handle Monitor again until test is complete!

## Results

### 8 5 MINUTES TO RESULTS

Display counts down

QC test result\*

Result

# of tests left

This result cycle remains displayed for 60 minutes or until the next Test Cartridge is inserted.

\* If “QCOK” is not displayed, please see list of error codes on reverse side.

**If you cannot resolve an error, please call Technical Support at 1-877-212-4968 x1.**

### 9 REUSE MONITOR

Save Monitor

Discard Test Cartridge

**THE MONITOR IS REUSABLE**

To run another test, use a new Sampler and Test Cartridge from the same kit and return to Step 1, “PREPARATION.”

**Product Dating & Lot ID Label**

**ALWAYS MATCH LOT NUMBERS**

Use Monitor only with the materials included in the original kit. The Monitor will expire after the programmed number of tests have been run. If another Test Cartridge is inserted, the Monitor will display “00 TL.”





1. Buris, C.A., Ashwood, E.R. *Tietz Textbook of Clinical Chemistry*, 3rd Edition, W.B. Saunders Co., 1999.
2. Nathan, D.M., et al. *The clinical information value of the glycosylated hemoglobin assay*. N Engl J Med 1984; 310: 341-346.
3. The Diabetes Control and Complications Trial Research Group. *The effect of intensive treatment of diabetes on the development and progression of longterm complication in insulin-dependent diabetes mellitus*. N Engl J Med 1993; 329: 977-986.
4. American Diabetes Association. *Standards of medical care for patients with diabetes mellitus*. Diabetes Care. 1999; 22 (suppl 1): S32-S41.
5. Fogh-Anderson, N., D'Orazio, P. *Proposal for standardizing direct-reading biosensors for blood glucose*. Clin Chem 1998; 44(3): 655-659.
6. MLO Supplement. *Point-of-Care Testing*, 1992.
7. Cagliero, E., Levina, E.V., Nathan, D.M. *Immediate feedback of A1C levels improves glycemic control in type 1 and insulin-treated type 2 diabetic patients*. Diabetes Care 1999; 22(11): 1785-1789.
8. Goldstein, D.E., Little, R.R., Wiedmeyer, H.M., et al. *Glycated hemoglobin: Methodologies and clinical applications*. Clin Chem 1986; 32: B64-B70.
9. American Diabetes Association: *Clinical Practice Recommendations 2006*. Diabetes Care, 2006; 29 (Suppl. 1).

**Bayer HealthCare, LLC**  
510 Oakmead Parkway  
Sunnyvale, CA 94085-4022  
tel 1 877 212 4968 x1  
fax 408 524 6595  
[www.A1CNow.com](http://www.A1CNow.com)