

A Rapid Point-of-Care Test for Acute Respiratory Infections

Introducing FebriDx[®], a new rapid POC assay to aid in the diagnosis of acute respiratory infection and differentiate bacterial from non-bacterial etiology.



- Identify a bacterial infection and initiate appropriate treatment during a patient visit
- Improve practice workflow by identifying which patients require attention
- Ensure that bacterial infections are not overlooked
- Easily implement an outpatient antimicrobial stewardship solution


- ✓ **99% negative predictive value to rule out bacterial infections¹**
- ✓ **Instrument-free**
- ✓ **Results after 10 minutes from a fingerstick**
- ✓ **FDA cleared, easy-to-use procedure**




Wilburn Medical, Inc.
WilburnMedicalUSA.com
1-877-945-2876




Simple Instrument-Free Test Procedure

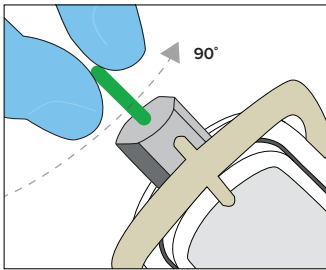


1 Prepare Lancet

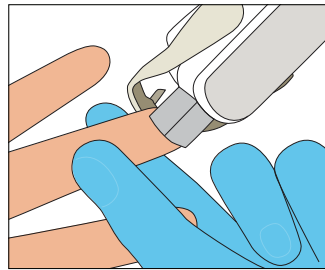


2 Lance Finger

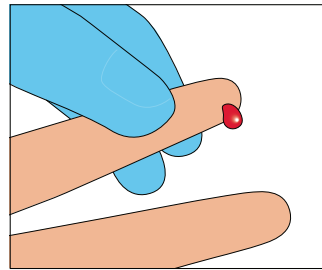


3 Massage Finger

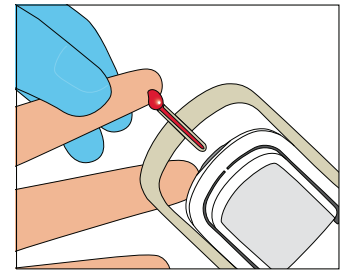


4 Collect Blood

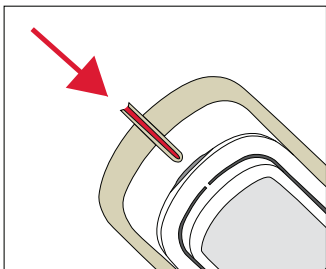


5 Verify Tube Is Full

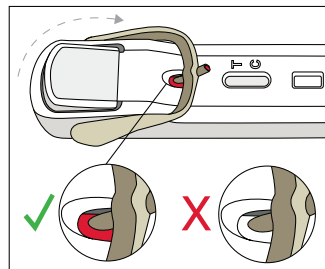

6 Transfer Blood

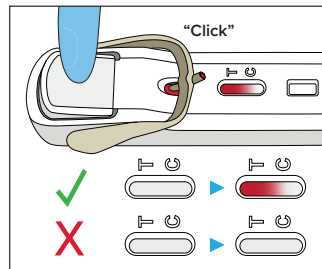


7 Deliver Buffer

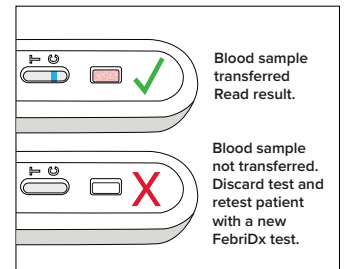


8 Check Blood Transfer

A **blue** control line must appear in the result window for the test to be valid.


Bacterial Infection

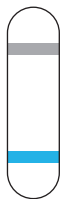

Non-Bacterial Etiology

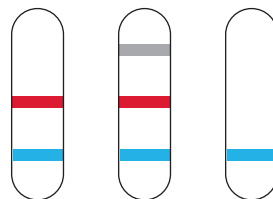

Invalid

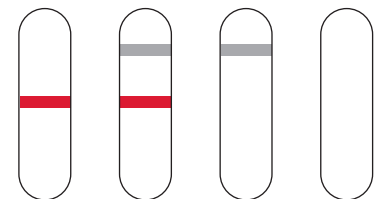

PATIENTS WITH SYMPTOMS OF ACUTE RESPIRATORY INFECTION
(e.g., fever, cough, shortness of breath, cold, sore throat)

Wilburn Medical, Inc.
WilburnMedicalUSA.com
1-877-945-2876

1. Shapiro NI, Filbin MR, Hou PC, et al. Diagnostic Accuracy of a Bacterial and Viral Biomarker Point-of-Care Test in the Outpatient Setting. *JAMA Network Open*. 2022;5(10):e2234588-e2234588. doi:10.1001/jamanetworkopen.2022.34588